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USER GUIDE FOR THE BLANKET CUSHION BY REBACARE®

- Before using the Blanket cushion, it is important to clean the outside with 1000 ppm chlorine, 70% alcohol, OxyFoam, OxyWipe or detergent. See also the guidelines of the Infection Prevention Working Group (WIP) (www.rivm.nl)
- Ask your occupational therapist or physiotherapist how to correctly position the Blanket cushion.
- Blanket cushion cannot be washed in a washing machine or tunnel washer.
- For an explanation of the different applications, please refer to our website: www.rebacare.com/instructions
- Blanket cushion is not suitable for children aged 3 and under.
- Blanket cushion does not contain any latex and can be disposed of with regular household waste.
- Please ensure that the patient indicates that the Blanket cushion feels comfortable!

Blanket cushion

- Place the Blanket cushion at the end or side of the mattress. If the client is shorter than 1.85 meters in length, you can place a pillow between the Blanket cushion and the back of the bed. For taller clients (taller than 1.85 meters) you can use the bed's bed extender.

Blanket cushion:

- Disinfect with 70% alcohol and max. 1000 ppm Chlorine, Oxyfoam, detergent or OxyWipe.
- Easy to store and will not damage beds or mattresses.
- Please also watch our useful instruction video: www.rebacare.com

MEDICAL PRODUCT CAN BE REUSED AFTER CLEANING

CHECK THE PRODUCT FOR ANY DAMAGE

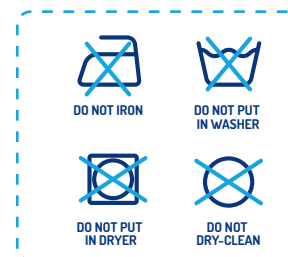
DISPOSE OF THE PRODUCT IF DAMAGED

DISINFECT WITH 70% ALCOHOL AND MAX. 1000 PPM CHLORINE, OXYFOAM, DETERGENT OR OXYWIPE

WIPE THE PRODUCT WITH A CLOTH MOISTENED WITH WATER

REBACARE® GIVES A 2-YEAR PRODUCT GUARANTEE

After a two-year period of intensive use in terms of cleaning and hygiene (incontinence), REBACARE® recommends that the product is regularly inspected and replaced if necessary.

Blanket cushion

Image 1

Image 2

WARNING:

- Blanket cushion is only suitable for use on the feet.
- Any serious incident related to the patient device must be reported to REBACARE® and the competent authority of the member state where the patient resides.
- Do not combine parts with other positioning systems.
- Any wound treatment of the patient should be performed before using the Blanket cushion.

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