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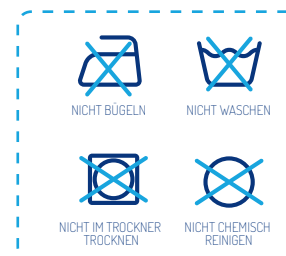
USER INSTRUCTIONS FOR THE DYNAMIC POSITIONING ORTHOSIS FREE EAR EAR PILLOW BY REBACARE®

The innovative Free Ear ear pillow by REBACARE® leaves the ear free.

- Before using the Free Ear ear pillow, it is important to clean the outside with 1000 ppm chlorine, 70% alcohol, OxyFoam, OxyWipe or detergent. See also the guidelines of the Infection Prevention Working Group (WIP) (www.rivm.nl)
- The Free Ear ear pillow can not be washed in a washing machine or tunnel washer.
- For an explanation of the different applications, please refer to our website: www.rebacare.com/instructions
- The Free Ear ear pillow is not suitable for children aged 3 and under.
- The Free Ear ear pillow does not contain any latex and can be disposed of with regular household waste.
- Intended use: leaves the ear free of users/patients (over 3 years old) to prevent/treat pressure ulcers in the following indications: Ear decubitus, Chondrodermatitis nodularis helices, Oxygen tube over the ear, Tube-feeding or mask over the face, Hearing aid and Tube prone pillow during ventilation.
- Place the patient's head on the Free Ear ear pillow so that the ear is positioned in the cut-out area.
- In the image of the Free Ear ear pillow shown here (see image) the cushion is suitable for the left ear.
- To use the Free Ear ear pillow of the patient for the right ear, the Free Ear ear pillow has to be turned from right to left.
- Ensure that the face of the patient is directed toward the cut-out part.
- The patient's neck is on the narrowest part of the Free Ear ear pillow.
- The Free Ear ear pillow is not secured, but is loose on the bed.
- REBACARE® recommends changing the position of the patient every 3 hours, and if there are no medical objections, to turn the patient over so that the other ear can be positioned on the Free Ear ear pillow.

Free Ear ear pillow

REBACARE®
Sasker van Heringawei 12
9034 GD Marsum

The Netherlands
info@rebacare.com
www.rebacare.com

Free Ear
ear pillow

MEDICAL PRODUCT CAN BE REUSED
AFTER CLEANING

CHECK THE PRODUCT FOR ANY DAMAGE

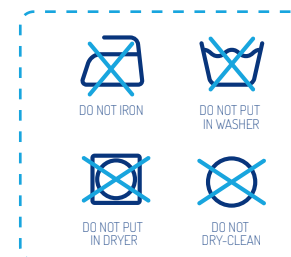
DISPOSE OF THE PRODUCT IF DAMAGED

DISINFECT WITH 70% ALCOHOL AND MAX.
1000 PPM CHLORINE, OXYFOAM,
DETERGENT OR OXYWIPE

WIPE THE PRODUCT WITH A CLOTH
MOISTENED WITH WATER

REBACARE® GIVES A 2-YEAR PRODUCT GUARANTEE

After a two-year period of intensive use in terms of cleaning and hygiene (exudate), REBACARE® recommends that the product is regularly inspected and replaced if necessary.

REBACARE® gives a 6-month guarantee on the covers. These covers must be washed before use in accordance with the washing instructions. REBACARE® has 2 different matching covers:

Standard Nicky stretch covers (80% cotton and 20% polyester, can be washed at 60 degrees Celsius) can be put in the dryer

Stretch covers (100% cotton, can be washed at 60 degrees Celsius) can be put in the dryer

WARNING:

- Due to possible leakage of the contents (polystyrene beads and/or siliconized hollow fiber) to the environment, breathing problems may occur in the event of possible oral or nasal ingestion. To prevent this, only the REBACARE® models may be used with the original REBACARE® covers. You can order these original covers.
- Make sure that the patient (if possible) is in a 30-degree position when lying on his side. This prevents the patient from turning, so that unwanted prone positioning is prevented.
- The product for the head is not suitable (contains polyurethane ticking). **Therefore they are not suitable for people with epilepsy.**
- Any serious incident related to the patient device must be reported to REBACARE® and the competent authority of the member state where the patient resides.
- Any wound treatment of the patient should be performed before using the product.

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