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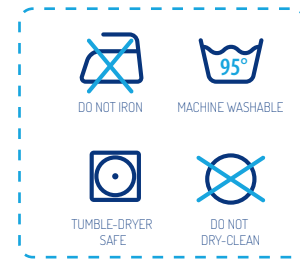
Die Niederlande
info@rebacare.com
www.rebacare.com

Turnability

USER MANUAL TURNABILITY BY REBACARE®

- Turnability should be washed at 95 °C before use. See also the guidelines of the Infection Prevention Working Group (WIP) (www.rivm.nl)
- Turnability does not contain any latex and can be disposed of with regular household waste..

Intended use of Turnability:

- Positioning users/patients (over 3 years old) to prevent and/or treat decubitus in the following indication(s): paraplegia, CVA, ALS, MS, Parkinson's disease, congenital abnormalities, comfort, dementia and/or conditions where postural management is desired, for example in Intensive Care Units (ICU), nursing homes, hospice or at home.
- Patients who have difficulty turning over/onto their side themselves due to decreased strength.
- For patients who are too difficult for health professionals to turn over/onto their side due to illness or incapacity.
- For patients who are incontinent and would like to keep the bed dry.
- As a solution for patients who want to keep a sliding sheet in bed, which is against the guidelines.
- Position the patient on top of Turnability in such a way that the patient's shoulder is properly positioned on it.
- You can pull the outer handles to move the patient from right to left (or from left to right).
- Place Turnability on the bed with the slide tunnel at the bottom.
- The middle handle is used to (if necessary) correct the tensile stress on the skin that has been created. The middle handle can also be used to straighten the hip (femoral head straight in the hip socket).
- Make sure that patient is placed on the quilted pattern side.
- Make sure the slide tunnel is not folded under the Turnability mat.
- Turnability is also extremely useful for persons with reduced strength who would like to turn over/onto their side in bed themselves.

REBACARE® GIVES A 6-MONTH PRODUCT GUARANTEE

After a six-month period of intensive use in terms of cleaning and hygiene (incontinence), REBACARE® recommends that the product is regularly inspected and replaced if necessary.

WARNING:

- **Only use Turnability in combination with a passive lift.**
- **Do not use Turnability in combination with a standing lift or without a passive lift due to the risk of slipping.**
- **Make sure that the patient is placed on the bed with the passive lift. Do not use the Turnability handles to hang from the passive or transfer lift.**
- **Turnability is not suitable for children under 3 years of age or for use without supervision.**

Top of Turnability

Bottom of Turnability

Pull the two outer handles to
move the patient.

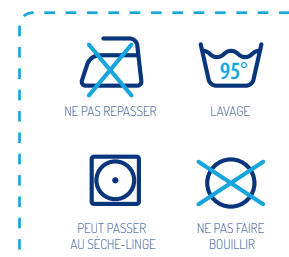
The middle handle is used to (if
necessary) correct the tensile
stress and straighten the hip.

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